

Descosept Spezial



Application: According to the Medical Devices Directive (MDD): Disinfection and cleaning of non-invasive medical devices sensitive to alcohol. According to the Regulation of biocide products: Disinfection and cleaning of medical inventory and all kinds of surfaces sensitive to alcohol, also in the food industry and restaurants.

Instructions: Wipe the surface of the device until it is completely not get wet. When the time of

exposure, if necessary, wipe the surface with a cloth disposable.

The spectrum of activity and exposure time: The bactericidal effect of including MRSA,

fungicidal effect: undiluted 1 min inactivate viruses with enveloped according to RKI

recommendations, including BVDV, HBV / HCV / HIV and Influenza virus: undiluted 30 sec

Rota viruses: undiluted 30 sec Papova- / Polyoma Virus: undiluted 2 min

Composition: 100 g solution contains: 0.25 g Didecildimetilamonijumchlorid 0.5 g quaternary ammonium compound benzyl-C12-16-alkildimetilchlorid

CAUTION In case of contact with eyes: Rinse cautiously with water for several minutes. If the present contact lenses, remove them if possible. Continue rinsing. Cancel OK contents / container in accordance with local and national regulations.