

## Thermoton Cleaner

**Dr. Schumacher**

### THERMOTON CLEANER

[www.sinofarm.co.rs](http://www.sinofarm.co.rs)

**THERMOTON® CLEANER**  
alkalischer Flüssigreiner

Art. No. 10747  
USA: 10747

Inhalt  
e 5L

CE mark  
ISO 9001  
ISO 14001

**Dr. Schumacher GmbH**  
Postfach 1010, 31131 Hannover  
Tel.: 0511 4100-100  
www.schumacher.com

Application: Cleaning at 93oC thermostable instruments and materials and delicate cleaning of thermolabile instruments such as anesthetic probes, rubber, synthetic materials or elastomers at 60oC. For cleaning program sectors A and B according to RKI (Robert Koch Institute) as well as for Vario TD programs.

Directions: Place the instruments-if possible immediately after use in a machine according to the

instruction. Every surface and cavity must be accessible washer fluid.

Dosage: depending on the degree of dirt / water quality: 3-7ml / L Exposure time at 93oC: 10  
mn Ingredients: Potassium hydroxide, silicates, phosphonates