

Descoprent



Application: The prepared solution for rapid disinfection and thorough cleaning of dental work instruments and prosthetic dental materials such as polyethers, A / C silicones, alginates and hydrocolloids.

Use: For a short dive objects of alginate and hydrocolloid and allow to dry for 10 minutes.

Immerse all other materials in solution for disinfection (according to the exposure time) and then

rinse with water.

Exposure time:

The bactericidal effect of including MRSA, fungicide according to DGHM / VAH directives (conditions of contamination) nerazblažen- 5min inactivates enveloped viruses from including HBV, HIV, BVDV, HCV, Vaccinia, In fl uenza virus: unadulterated-1min Rota virus: nerazblažen-5min

Composition: 100 g solution contains: pentanediol 0.01 g Ethanediol 0.20 g

Dimetilbenzilalkilamonijum chloride 0.48 g

Additional information: In case of contact with eyes: Rinse cautiously with water for several minutes. If the present contact lenses, remove them if possible. Continue rinsing. Cancel OK contents / container in accordance with local and national regulations. Keep out of the reach of children.