

## Perfektan Neu



Fields of application: Disinfection and cleaning of surgical instruments. Ideal for disinfecting pre-cleaning of all types of surgical instruments. Suitable for use in an ultrasonic bath.

Instructions: Immerse opened instruments into the working solution immediately after use. Every surface and cavity must be completely soaked working solution. After exposure time, rinse instruments well with clean water and allow to dry. The final rinse is recommended to be done

with distilled water.

Concentrations and range of activities: Exposure time according to the guidelines of the DGHM / VAH tested in terms of impurities (including TB, MRSA): 1% -60min .; 2% -30min .; 3% -15 min.

Active against all viruses with membrane (Vaccinia, BVDV, HBV, HIV, HCV) according to RKI recommendations: 1% -60min .; 2% -30min .; 3% -15 min.

Product: PERFECT NEU Manufacturer: Dr.Schumacher GmbH, Am Roggenfeld 3, D-34323 Malsfeld-Beiseforth, Germany Importer / Distributor / The holder of registration: Sinofarm Ltd. Koste Nadja 31 11160 Belgrade, Serbia [www.sinofarm.co.rs](http://www.sinofarm.co.rs) number of solutions ALIMS: xxxxxxxxx of xx / xx / xxxx Number: 00-136-010

Composition: 100 g contains 3.6 g of N- (3-aminopropyl) -N-dodecylpropane-1,3-diamine

CAUTION causes severe burns on the skin and damage to the eye. May cause allergic skin reaction. Long-term toxic effects on aquatic life. Wear protective gloves / protective clothing / eye protection / face protection. If swallowed: Call a poison control center or physician if you feel unwell. If it comes into contact with skin or hair: Take off immediately all contaminated clothing. Rinse skin with water. If it comes into contact with the eyes: Rinse cautiously with water for several minutes. If the present contact lenses, remove them if possible. Continue rinsing. Immediately call a poison control center or doctor. Store under lock and key.